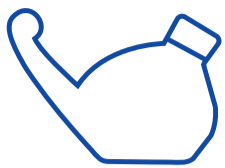
NASENDUSCHÉ Starter-Set

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Wächter's Naturheilmittel GmbH
Rheinzaberner Str. 8
DE-76761 Rülzheim

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Instructions for use – please read carefully!

EN **NASAL IRRIGATION SET**
With nasal irrigator and nasal saline rinse. To prevent colds and protect against high pollen counts.

Dear user,

Please read the following instructions carefully as they contain important information about what you should be aware of when using this medical device.

Contents:

1 nasal irrigator + 50 sachets of nasal saline rinse each weighing 2.25g

Intended purpose and application:

For simple and intensive cleaning and moistening of the nasal mucosa

- as a preventive and supportive measure for colds
- when the pollen count is high
- when exposed to high levels of dust and dirt

For adults and children from 3 years.

Contraindications:

Frequent and heavy nosebleeds, severe injuries of the nasal cavities with direct connections to the fluid cavities of the brain.

Information for special patient groups:

If the instructions are followed, the nasal saline rinse can be used during pregnancy and while breast-feeding as well as for children from 3 years and older people.

Side effects:

Occasionally there is a tingling and burning sensation in the nose if the nasal saline rinse is not properly dissolved, and headaches have been observed in isolated cases immediately after rinsing. No countermeasures are required.

Side effects of misuse:

If used incorrectly, especially if the concentration of the rinsing solution is too high or too low when rinsing the nose, the nasal mucosa can become irritated or dry. Oral ingestion of the salt can lead to stomach and circulatory problems with a temporary increase in blood pressure.

Dosage:

Nasal rinses can be carried out up to 3 times a day with 250ml of solution each time. If used as intended, rinsing with the nasal irrigator and nasal saline rinse can be carried out for an unlimited time. The filling volume of the nasal irrigator is 250ml.

Please note:

The water required to produce the rinse solution must not be warmer than 50°C, must be of at least drinking water quality, and should always be fresh. In special circumstances, such as patients who have mucoviscidosis (cystic fibrosis) or who have had surgery in the previous 3 days, or if the quality of the drinking water is unknown, sterile or boiled water should be used.

How to use the Nasal Irrigation Set:

The nasal irrigator should be rinsed thoroughly with warm water before each use.

Fig. 1

Fig. 2

The rinsing pressure drops as the amount of rinsing solution in the nasal irrigator decreases. To ensure sufficient rinsing pressure until the irrigator is completely empty, close the opening in the cap again with your index finger and carefully press the side walls of the nasal irrigator.

More information about using the device:

If your nose is very congested, the rinsing pressure can be increased and adjusted to your needs by pressing on the nasal irrigator and closing the opening in the cap at the same time. After irrigation, turn your head to each side so that all of the irrigation solution flows out of the nasal cavity. Do not blow your nose during or immediately after rinsing, otherwise loosened secretion could be forced into the sinuses and the middle ear.

Cleaning and storage instructions:

Before and after each use, the nasal irrigator should be thoroughly rinsed out with warm water. For storage, dry the nasal irrigator well and store it upside down until it is completely dry, as no liquid should remain in the container (risk of germs).

For intensive cleaning, the nasal irrigator can be cleaned in the dishwasher and placed in a baby bottle vaporizer. Please ensure that the highly elastic plastic material is not pressed during cleaning, as this will severely deform the material. Frequent heat treatment will change the shape and elasticity of the plastic material and must be avoided. Heat treatment over 100°C will destroy the device. Limescale removers suitable for coffee machines can be used to remove limescale residues. Please then thoroughly rinse the device several times with clear water.

Composition of the nasal saline rinse:

100% sodium chloride Ph. Eur. (pharmaceutical table salt), contains no preservatives and / or flow regulators.

Other information:

Please only use the nasal irrigator for nasal rinsing. In particular, please do not fill the nasal irrigator with solutions that are not suitable for nasal rinsing. The nasal irrigator is a simple medical device with a limited durability and shelf life. For hygiene reasons, it should be replaced every six months or after a maximum of 100 uses. Do not use nasal saline rinse after the expiry date printed on the package. Always use the prepared solutions when they are fresh and never store them: the nasal saline rinse does not contain any preservatives, so the solutions can quickly become contaminated (possible risk of infection!). Please always keep out of reach of children.

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