NASENSPÜLSALZ

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 Wachter's Naturheilmittel GmbH
Rheinbacher Str. 8
DE-76761 Rülzheim

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Instructions for use, please read carefully!

Medical device

EN NASAL SALINE RINSE to prevent colds and protect against high pollen counts

Dear user!
Please read the following instructions carefully as they contain important information about what you should be aware of when using this medical device.

Contents:
75 sachets of nasal saline rinse each weighing 2.25 g

Intended purpose and application:

For simple and intensive cleaning and moistening of the nasal mucosa

- as a preventive and supportive measure for colds
- when the pollen count is high
- when exposed to high levels of dust and dirt

For adults and children from 3 years of age.

Contraindications:
Frequent and heavy nosebleeds, severe injuries of the nasal cavities with direct connections to the fluid cavities of the brain.

Information for special patient groups:
If the instructions are followed, the nasal saline rinse can be used during pregnancy and while breast-feeding as well as for children from 3 years and older people.

Dosage and mode of administration:
Nasal saline rinse is used to prepare a nasal rinsing solution. For this purpose, the contents of a sachet of nasal saline rinse are dissolved in 250 ml of warm water (drinking water quality), and the solution is then used to rinse the nose. Please shake or swirl the irrigator vigorously to ensure that the salt is completely dissolved - if the nasal saline rinse is not completely dissolved, it can lead to irritation such as a burning or tingling sensation.

Please note:
The water required to produce the rinse solution must not be warmer than 50 °C, must be of at least drinking water quality, and should always be fresh. In special circumstances, such as patients who have mucoviscidosis (cystic fibrosis) or who have had surgery in the previous 3 days, or if the quality of the drinking water is unknown, sterile or boiled water should be used. Nasal rinses can be carried out up to 3 times a day with 250 ml of solution each time. Please use a standard nasal irrigator with a filling volume of 250 ml. If used as intended, nasal rinses with nasal saline rinse can be carried out for an unlimited time.

Side effects:
Occasionally there is a tingling and burning sensation in the nose if the nasal saline rinse is not properly dissolved, and headaches have been observed in isolated cases immediately after rinsing. No countermeasures are required.

Side effects of misuse:
If used incorrectly, especially if the concentration of the rinsing solution is too high or too low when rinsing the nose, the nasal mucosa can become irritated or dry. Oral ingestion of the salt can lead to stomach and circulatory problems with a temporary increase in blood pressure.

Storage instructions:
Always use the prepared solutions when they are fresh and never store them: nasal saline rinse does not contain any preservatives, so the solutions can quickly become contaminated (possible risk of infection!). Do not use nasal saline rinse after the expiry date printed on the package. Please always keep out of reach of children.

Composition:
100 % sodium chloride Ph. Eur. (pharmaceutical table salt), contains no preservatives and / or flow regulators.

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